



NON-STERILE: FOR SINGLE PATIENT USE ONLY



NEW TAMPER  
PROOF PACKAGING



Manufactured for EMSI

[Product Name Here]  
Conductive Garment

Features:

- Item #1
- Item #2
- Item #3





# important information about your new conductive electrode

## Instructions For Use

1. Remove all jewelry from the area covered by garment.
2. Clean skin thoroughly with soap and water.
3. Insert 0.080" pin from active lead wire (red) of the stimulator into electrode connector on garment so that no metal is visible.
4. Place garment over area to be treated with the electrostimulation – make sure the garment conforms snugly to skin. It is recommended to apply electrolyte solution on all stimulation fabric prior to use – consult your practitioner.
5. Insert 0.080" pin from dispersive lead wire (black) into electrode connector on dispersive.
6. Place dispersive electrode on the appropriate portion of the body as per practitioner's protocol. Do not place directly over spine. A second garment can perform as a dispersive electrode. If another garment electrode is used as a dispersive electrode, placement of the dispersive electrode should ensure trans-thoracic stimulation is not possible.
7. After understanding the operation of the stimulator being used, turn stimulator on and adjust as directed by practitioner.
8. Exercising the range-of-motion of treatment area is recommended during stimulation.

## Care And Cleaning

1. Hand wash in cold water.
2. Air dry only – do not use a dryer or iron.
3. Do not bleach.



Copyrights © 2009 EMSI. All rights reserved. EMSI, the EMSI logo, and [product name] are trademarks and/or registered trademarks of Electrostim Medical Services, Inc. All other marks belong to their respective holder. DOC NO. PLB-C717 REV A